

# Dental Water Washable Model Resin 3D Printing Material

RF-ZJ-02



- Biocompatible
- Fast forming rate
- High precision
- Support OEM/ODM servies



Material features fast forming speed and high forming precision Its high degree of detail reproduction and good high-temperature resistance can meet the requirements for the production of high-temperature dental models. The stable material structure can withstand the high-pressure and high-temperature impacts of hot-pressing molding instruments.

<b>Applications:</b>	Dental model	<b>Color:</b>	Milky White, Grey	<b>Material:</b>	Resin
<b>Applicable printer type:</b>	DLP and LCD 3D printer		<b>Characteristics:</b>	High strength、 water washable High elasticity、 high-temperature	

Physical properties	Testing method	Typical value
Density	ASTM D792	1.05-1.15 g/cm3
Viscosity	ASTM D445	400 mPa ·s
Hardness	ASTM D2240	80-85 ShoreD

Mechanical properties		
Tensile Strength	ASTM D638	57.6 MPa
Elongation at Break	ASTM D638	25 %
Flexural Strength	ASTM D790	49 MPa
Flexural Modulus	ASTM D790	1200 MPa

Note: above parameters are for reference only. The performance of cured materials will be affected by factors such as equipment, environment, parameter settings, post-processing methods, and testing methods, which will cause differences. Please contact us for professional advice.

Print parameters

Representative Machine	Exposure Time/s	Bottom Exposure Time/s	Bottom Layer Count	Lift Distance /mm	Lift Speed /mm • min-1	Retract Speed /mm • min-1
Phrozen sonic mini 8K S	5	20	3	4+4	40+120	180+60
Elegoo saturn 4 Ultra	3	25	3	Default standard parameters of the device		
Elegoo saturn 3 Ultra	3	25	3	4+4	60+240	60+90
Creality HALOT-MAGE Pro	1.5	10	3	8+2	180+180	180+180
Anycubic Photon Mono 4 Ultra	4	25	3	4+4	120+180	240+180

Note: The list's brands are solely owned by the respective brand owners. Here is only description provided

Post-Processing Procedure and Note

- 1.The model can be cleaned using isopropyl alcohol in the ultrasonic cleaning machine, and try not to use high-frequency shock or force brushing the model to avoid damage to the surface details of the model.
- 2.Thoroughly blow the model dry with a hair dryer or the like;
- 3. It is recommended to remove the support for model with supports first, and then post-cure treatment. If you remove the supports after it' s been post-cured , it will easily cause damage to the contact surface of the support point;
- 4. For some occasions where certain toughness is required, you can choose to cure with UV lamp for 5 minutes. The printed parts should be kept in a cool dry place.

Safety Precautions

- 1.Eye Contact: Immediately flush with plenty of clean water (under eye lids) for at least 20 minutes. Hold eyelids apart to ensure flushing. Washing within one minute of contact is essential to achieve maximum effectiveness. Seek medical attention immediately.
- 2.Skin Contact:Remove contaminated clothing and rinse contact area thoroughly with soap and water.
- 3.3D resin is not approved for use with food, drink, or medical application on the human body
- 4.For additional information please see the Material Safety Data Sheet.

Safety Precautions

Please store in a cool place below 25°C(77°F), away from direct sunlight. Ordinary visible light may cause the resin to polymerize and gel.



3D Printing Product **Manufacturer**  
Equipment Automation **Supplier**  
Your 3D Printing technology **Partner**

Follow us: RayForm



## Office

### Shenzhen RayForm Technology CO., LTD.

Add: Building 3 Westside, Guole Science Technology Park, No.1 Lirong Road, Dalang Street, Longhua District, Shenzhen 518100, CHN.

TEL: +86-755-27615363

Emial: [contact@rayformtech.com](mailto:contact@rayformtech.com)



## Factory

### Ziyang RayForm Technology CO., LTD.

Add: 5th Floor, Unit 9, Building 3, No.222, West Third Section of Outer Ring Road, Yanjiang District, Ziyang City, Sichuan, CHN